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Intended use:

The Kneefix Brace is an orthosis for immobilising the knee joint in an extended position or in slight flexion.

Indications:

Injuries to joints, muscles or ligaments
Preoperative, postoperative, post-traumatic immobilization of the knee joint

Contraindications:

Currently unknown.

Application instruction:

Open all Velcro fasteners and straps. The circumference of the rail can be lengthened or shortened. To do this, loosen the Velcro fasteners of the side parts and reattach the side parts at the corresponding position (1). When positioning the side parts, make sure that the kneecap is not cut out. Place the splint around the leg and adjust the circumference if necessary as described above. Position the product so that the leg is centered in the middle section and the patella is in the recess of the splint (2+3). Close the lower leg straps with Velcro, starting with the strap directly below the knee joint. Then close the thigh straps with Velcro, starting with the strap directly above the knee joint (4).

Application note / Caution:

- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:

The product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product (if existent) should not be washed and must be removed if possible. These parts can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:

Polyester, Nylon, Polypropylene (PP), Aluminium

Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:

Disposal via household waste is possible.

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Composition des matériaux:

Polyester, Nylon, Polypropylène (PP), Aluminium

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Materialsamenstelling:

Polyester, Nylon, Polypropyleen (PP), Aluminium

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