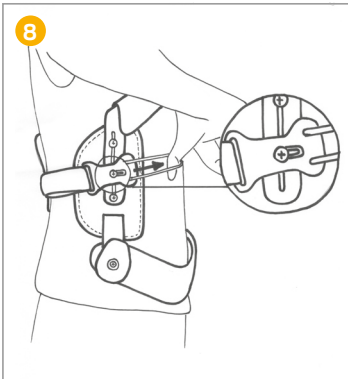
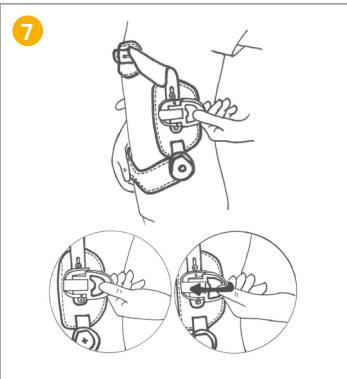
Size	S	M	L	XL
Hip circumference (cm)	60-75	75-90	90-105	105-115
Body height (cm)	SC37K: up to 160 SC37: from 160			

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EN
Intended use: Orthosis for stabilization and movement restriction of the lumbar and thoracic spine.
Indications: Stable vertebral compression fracture of the lower thoracic spine and the lumbar spine Scheuermann's disease Temporary, postoperative immobilization
Contraindications: Currently unknown.

Application instruction:
First set the height and width of the product.
To adjust the height of the product, loosen the screws on the two side rods (1). Set the desired height and fix the screws firmly in the appropriate positions. To adjust the width of the pro-duct, loosen the screws on the horizontal bar of the chest pad (2). Set the desired width and fix the screws again firmly in the appropriate positions. Open the back belt. To do this, press the blue sliding button backwards and open the grey buckle. Now loosen the quick release by pulling it forward and thus releasing it from the fastening. Place the product around the upper body and fasten the quick release fastener again (3+4). Position the back plate in the middle of the back and close the back strap tightly with the Velcro fastener (5). Close the grey buckle (6). To put the product down, first open the grey buckle by pushing the blue sliding button backwards (7). Then open the quick release fastener (8).

Application note / Caution:

- It is recommended to wear the product over a shirt.
- Change the settings of the product only after consultation with your treating physician.
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that no fabric parts are damaged by existing Velcro fasteners.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:
The product can be cleaned by hand with a mild detergent. Plastic and metal parts of the product (if existent) must not be washed and must be removed if possible. These parts can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to prevent damage to other laundry items. Leave to air dry. Avoid direct heat sources and do not machine dry or iron. Do not use chemical cleaning products, bleaches, solvents or fabric softeners. Fatty or acidic substances, ointments or lotions can damage the material.

Material composition:
Aluminium, Polyethylene (PE), PU foam, Steel, Nylon, Polyurethane (PU)

Storage instructions:
Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:
Disposal via household waste is possible.

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