DK

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**Intended use:**

Orthosis for immobilising the elbow joint.

Orthosis for immobilising the elbow joint

**Indications:**

Injury of the ligaments in the area of the elbow

Elbow sprain

Joint effusions

Posttraumatic, postoperative immobilisation of the elbow joint

Orthosis for immobilising the elbow joint

**Contraindications:**

Currently unknown.

### Application instruction:

Open all Velcro fasteners and straps. Place the brace around the arm (1). Position the product so that the longer padding is on the forearm and the shorter padding is on the upper arm and the elbow is well placed in the bend of the brace. If necessary, you can mould the aluminium braces to the arm by bending them. First close the Velcro fastener just below the elbow, then the fastener just above the elbow and then all the remaining fasteners (2). Close the straps attached to the inside and outside in the elbow area using the Velcro fasteners to prevent the arm from straightening (3). To relieve strain on the injured arm, you can use the arm sling strap. To do this, place the strap with the padding around the neck (4). Thread the end of the strap through the plastic loop, place the forearm in the arm sling and close the Velcro fastener. The length of the strap can be adjusted using the Velcro fastener.

### Application note / Caution:

- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

#### Care instructions:

The product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product (if existent) should not be washed and must be removed if possible. These parts can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

#### Material composition:

Polyester (PES), PUR foam, Cotton (CO), Polyamide (PA), Aluminium

#### Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

#### Disposal note:

Disposal via household waste is possible.

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