Funktionsbeskrivelse:

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Function description:

The set includes five heel wedges in different sizes. Depending on the shape and length of the foot, a maximum of three wedges can be placed on top of each other and can be fastened in the heel area of the orthosis.

The heel wedges serve for adjusting the clubfoot position.

Application instruction:

Select the wedges that fit the size of your foot. Use as many wedges as necessary to achieve the desired height or as necessary for the correct heel angle. However, only place a maximum of three wedges on top of each other. One wedge segment corresponds to a height increase of approx. 10 mm. Position the wedges as far back as possible in the heel area of the lower leg foot orthosis (1). Place the largest wedge at the bottom, the second largest on top of it and the smallest wedge on the very top. Position the wedges so that the heel rests comfortably and with as little pressure as possible. Adhesive strips are provided on the lower side of the wedges that can be used to stick the wedges on top of each other to prevent slipping (2). For this, remove the adhesive foil, position the wedges correctly on top of each other and firmly press the wedges against the adhesive area to ensure proper adhesion. Attach the bottom wedge to the heel area on the orthosis sole. In the course of the therapy, the number of wedges can be adjusted.

Application note / Caution:

- The heel wedge may only be used in connection with a matching orthosis.
- When using several wedges, make sure that these have been stably positioned on top of each other to prevent slipping.
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:

The product can be cleaned with a clean, damp cloth. Avoid direct heat sources and do not machine dry or iron. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:

Ethylene Vinylacetate Copolymer (EVAC)

Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:

Disposal via household waste is possible.

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