DK

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Intended use:

The band exerts targeted pressure on the area below the kneecap, thus relieving strain on the patellar tendon.

Indications:

Patellar tendinitis
Chondropathia patellae

Contraindications:

Currently unknown.

Application instruction:

Open the Velcro fasteners on both sides of the strap. Position the strap so that the silicone pad rests in the middle, directly below the kneecap. Pull both ends of the strap evenly towards the back and close the band using the Velcro fasteners (1+2).

Application note / Caution:

- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:

The product can be cleaned with a clean, damp cloth. Avoid direct heat sources and do not machine dry or iron. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:

Polyamide (PA), Elastane (EL), Polyester (PES), Polyethylene (PE), Silicone

Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:

Disposal via household waste is possible.

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