1

DK

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**Intended use:**  
Ankle orthosis to support and stabilize the ankle.

**Indications:**  
Distortion of the ankle, Ligament injuries, slight ligament instability, Supination trauma, Osteoarthritis, Irritation states in the ankle

**Contraindications:**  
Currently unknown.

**Application instruction:**  
Open all Velcro fasteners and straps. Place the orthosis around the foot. Position the product so that the heel is properly in the recess of the orthosis and the stabilization elements sit pressure-free. First, close the lowest strap on the back of the foot using the Velcro fastener. Then close the upper straps (1). Use the elastic band on the inside of your foot. Pull it across the back of the foot and fasten it with the Velcro fasteners above the ankle (2). Now use the elastic band on the outside of your foot. Pull it across the back of the foot (the already attached tape is crossed) and attach it with the Velcro fastener above the ankle (3).

**Application note / Caution:**

- It is recommended to wear a sock or stocking under the orthosis.
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

**Care instructions:**  
The product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product (if any) should not be washed and must be removed if possible. These parts can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

**Material composition:**  
Polyamide (PA), PUR foam, Cotton (CO), Polypropylene (PP), ABS plastic

**Storage instructions:**  
Store the product in a dry place and protect it from heat, direct sunlight and moisture.

**Disposal note:**  
Disposal via household waste is possible.

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