DK

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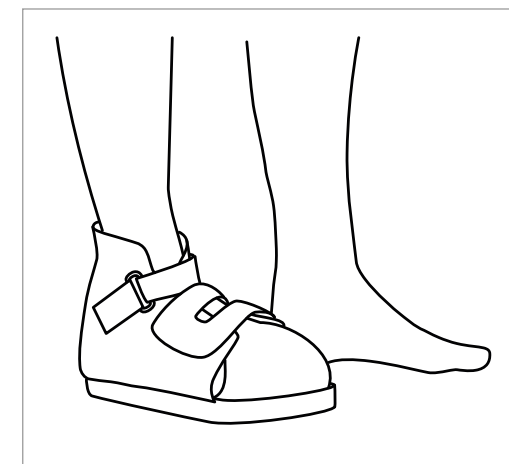
Size	XS	S	M	L	XL
Shoe size	33-34	35-37	38-40	41-43	44-46
Side	Universal				

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Intended use:

The bandage shoe is used when bandages need to be worn on the foot, for the protection of dressings and for stabilization.

Indications:

Protection of bandages in case of wounds and after surgery, Injuries in the forefoot area, Postoperative

Contraindications:

Currently unknown.

Application instruction:

Open the strap of the shoe and remove the Velcro strap on the back of the foot. Fold the side panels and the center part outwards. Place the foot in the open shoe (1). Make sure that the heel is at the back of the shoe and that the foot is securely positioned in the shoe. Place the middle part over the toes and the back of your foot and fold the side panels (2). First attach the wide part of the footback Velcro strap to the shoe and then close the two thinner parts of the Velcro strap with Velcro (3). Finally, close the strap using the Velcro fastener (4). If the belt is too long, the protruding part can be shortened.

Application note / Caution:

- While wearing the shoe, you are not allowed to drive a vehicle.
- In case of wear or damage to the sole of the shoe, the product should no longer be used.
- Do not wear the shoe directly on the skin.
- Walk slower than normal and be especially careful when walking on uneven paths and when climbing stairs.
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:

The product can be cleaned with a damp cloth and a mild detergent or with a soft brush. Before cleaning, close all Velcro fasteners to avoid damaging other textiles. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:

Polyester (PES), PUR foam, EVA, Polypropylene (PP), Polyamide (PA), TPR (Thermoplastic Elastomers)

Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:

Disposal via household waste is possible.

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