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**Intended use:**

Shoulder joint orthosis for light immobilization and relief of the shoulder joint.

**Indications:**

Immobilization and relief for post-traumatic conditions  
Light postoperative immobilization

**Contraindications:**

Currently unknown.

**Application instruction:**

Open the Velcro fastener of the shoulder strap. Place the arm of the injured shoulder in the pouch and position the arm so that the elbow lies well in the back of the pouch. Guide the shoulder strap behind the back over the healthy shoulder forward. Thread the strap through the loop on the wrist and close it using the Velcro fastener (1+2). Adjust the shoulder strap so that the arm of the injured shoulder is positioned approximately at right angles. Make sure that the shoulder pad is well positioned over the shoulder. If necessary, you can use the thumb loop as an additional support for the hand.

**Application note / Caution:**

- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

**Care instructions:**

The product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product (if any) should not be washed and must be removed if possible. These parts can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

**Material composition:**

Polyester (PES), Polyamide (PA), Polypropylene (PP), Cotton (CO), Polyurethane foam

**Storage instructions:**

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

**Disposal note:**

Disposal via household waste is possible.

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