DK

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Intended use:

Finger splint for immobilization of the middle finger joint (PIP joint).

Indications:

Buttonhole deformity, Joint capsule injuries, Stretching tendon injuries

Contraindications:

Currently unknown.

Application instruction:

Pull the splint over your finger (1). Position the splint so that the longer part (part A) is on top of the finger and the shorter part (part B) points in the direction of the fingertip (2). The front finger joint (DIP joint) should be freely movable. The splint should be fixed with a staple plaster or the optional Velcro strap for a better grip. Attach the fixation aid to the middle area of the upper part of the splint (part A) (3+4). When attaching the Velcro fastening tape or the staple plaster, make sure that sufficient ventilation holes remain uncovered.

Application note / Caution:

- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:

The product can be cleaned with a clean, damp cloth. Avoid direct heat sources and do not machine dry or iron. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:

High-Density Polyethylene (PE-HD)

Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:

Disposal via household waste is possible.

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