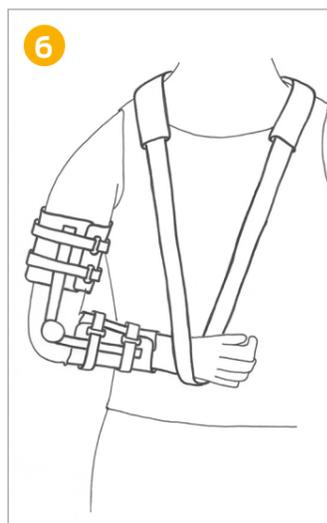
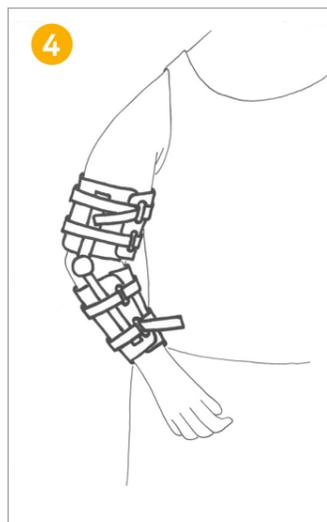
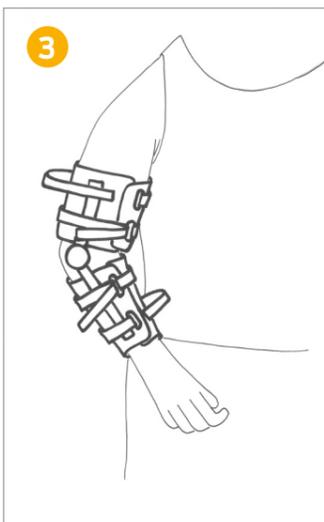
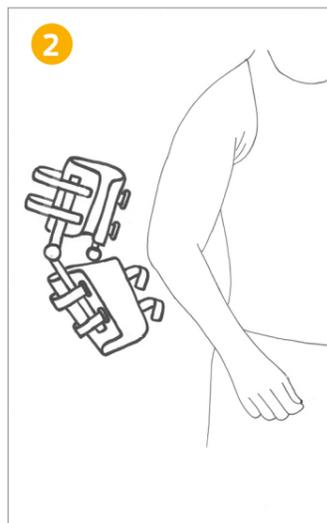
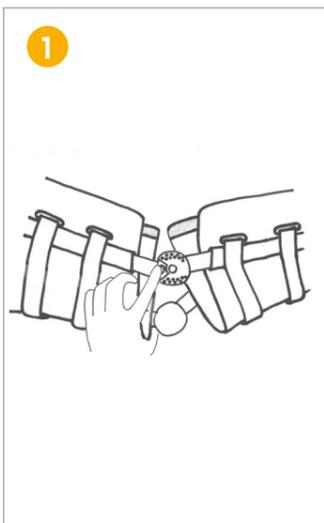
DK

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EN

Intended use:

Elbow orthosis to stabilise and guide the elbow joint with the option of restricting the range of movement of the joint.

Indications:

Instability in the area of the elbow joint
Soft tissue injuries
Stable fractures
Postoperative or posttraumatic mobilisation with restriction of movement of the elbow joint

Contraindications:

Currently unknown.

Adjusting the extension / flexion limit:

You can set the desired range of motion at the joints of the brace (1). To do this, press the silver locking pin down and turn the transparent cover of the joint until the opening lies above the desired limit pin. Remove the limit pin and insert it at the appropriate degree setting. Finally, turn the opening of the lid back over the locking pin and make sure that the locking pin fully engages. Flexion limit: 0° - 110° Extension limit: 0° - 90°

Application instruction:

Open all Velcro straps and the padding of the brace. Place the brace around the arm (2). Position the product so that the longer pad is resting on the forearm and the shorter pad is resting on the upper arm, and the joints are positioned at the elbow joint. Close the upper arm and forearm pad. First close the Velcro fastener directly below the elbow joint, next the fastener directly above the joint and then all the remaining fasteners (3+4). To relieve strain on the injured arm, you can use the arm sling. To do this, place the strap with the padding around the nape of the neck (5). Thread the end of the strap through the plastic loop, place the forearm in the arm sling and close the Velcro fastener (6). The length of the strap can be adjusted using the Velcro fastener.

Application note / Caution:

- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that no fabric parts are damaged by existing Velcro fasteners.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:

The textile parts of the product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:

Polyester (PES), PUR foam, Cotton (CO), Polyamide (PA), Aluminium

Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:

Disposal via household waste is possible.

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