1

DK

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Polypropylen (PP), Styrol-Ethylen-Butylen-Styrol (SEBS), Polyester (PES), PUR-Schaumstoff, Polycotton

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**Intended use:**

Ankle joint orthosis for stabilisation of the ankle joint.

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**Indications:**

Instability of the ankle joint, Capsular and ligament injuries, e.g., distortion (sprain), contusion (bruise), Pronation injury and supination trauma, Postoperative stabilisation after ligament suturing

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**Application instruction:**

Fully open the circular straps of the orthosis. Place the foot into the orthosis and put on the lateral plastic shells. Position the product in such a way that the curve of the heel belt points towards the back. Fit the heel belt to the foot so that the plastic shells fit well around the foot. To do this, open the outer Velcro fasteners on the shells and set the appropriate length (1+2). Ensure that the shells are centred on the lower leg side and sit well around the ankle. First, guide the lower and then the upper strap around the brace and fasten the straps using the Velcro fasteners (3+4). Should the straps be too long, the protruding part can be shortened with a pair of scissors.

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**Application note / Caution:**

- It is recommended to wear a sock or stocking under the brace.
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

**Care instructions:**

The textile parts of the product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

**Material composition:**

Polypropylene (PP), Styrene-Ethylene-Butylene-Styrene (SEBS), Polyester (PES), PUR foam, Polycotton

**Storage instructions:**

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

**Disposal note:**

Disposal via household waste is possible.

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