DK

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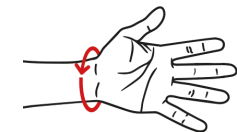
Model: SC019

Size	XS	S/M	L/ XL
Wrist circumference (cm)	11-14	14-17	17-20
Length (cm)	22		
Side	Universal		

Schaper Healthcare GmbH
Vogelweiderstraße 146 | 4600 Wels | Austria
office@schaper-healthcare.com
www.schaper-healthcare.com

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Intended use:

Wrist orthosis for the immobilisation of the wrist.

Indications:

Tendinitis (Tendovaginitis)
Carpal tunnel syndrome
Distortion of the wris
Posttraumatic and postoperative immobilisation

Contraindications:

Currently unknown.

Application instruction:

If necessary, you can take the aluminium splint out of the sewn-on pouch and shape it according to your wrist. Ensure that you place the aluminium splint correctly into the pouch afterwards, otherwise sufficient stability cannot be expected. Open all Velcro fasteners and straps. Pull the orthosis over the hand and position the thumb into the thumb opening (1). First close the middle strap over the wrist by guiding the strap around the orthosis and threading it through the sling made of synthetic material and closing it with the Velcro fastener (2). Thereafter, close the strap over the back of the hand and the circular strap on the forearm with the respective Velcro fasteners (3+4).

Application note / Caution:

- Regularly check the position of the aluminium splint so that it does not slip out of the pouch. If necessary, push the aluminium splint forward towards the fingers in order to achieve sufficient immobilisation of the wrist.
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:

The product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product (if existent) should not be washed and must be removed if possible. These parts can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:

Polyester (PES), Polypropylene (PP), Polyamide (PA), Aluminium, Polyoxymethylene (POM)

Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:

Disposal via household waste is possible.

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