Schaper Healthcare GmbH
Vogelweiderstraße 146 | 4600 Wels | Austria
office@schaper-healthcare.com
www.schaper-healthcare.com

Version V01/2021/04

DK

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Intended use:
Bandage with cold cushions and customizable compression for cooling and compression of the affected body part.

Indications:
All indications where cooling and optional compression are helpful, such as: Sports injuries and other traumas, Sprains (distortion), Bruises (contusion), Inflammation, Reduction of swelling, Postoperative

Contraindications:
Currently unknown.

Cooling:
Before use, place the cold pillow in the refrigerator or freezer for at least 2 hours. To do this, remove the cold cushion from the compression bandage.

Application instruction:
Remove the cold pillow from the fridge and fix it in the compression bandage using velcro fasteners. Connect the pump hose to the „ON/OFF“ valve. Place the product around the elbow and fix it using the hook-loop fasteners (1+2). Note that the cold cushion rests well on the affected area. To create compression, first turn the valve to ON (3). Then use the pump ball (4) to pump as much air until the desired compression is reached and turn the valve to OFF. If the valve is set to OFF, the pump hose can be removed without any air escaping. To drain air, reconnect the pump hose to the valve, turn the valve to ON, and press the black button on the pump ball. After use, drain the air completely from the compression bandage, clean the cold cushion and cool it again for the next use.

- Application note / Caution:**
- We recommend a duration of use of 10 - 20 minutes.
 - In case of cold sensitivity, use an intermediate cloth.
 - The cold cushion must not come into contact with coarse or sharp surfaces.
 - Do not heat. The product is not suitable for heat therapy.
 - Do not put the compression bandage, hose, nor pump in the refrigerator or freezer.
 - Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
 - This product is intended for use on a single patient only.
 - If necessary, the help of a second person can be beneficial for the correct application of the product.
 - Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
 - The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
 - Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
 - Do not put on the product too tightly, as this may affect the blood circulation.
 - If symptoms persist or worsen, please consult your medical doctor.
 - Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
 - If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
 - Do not make any changes to the product.
 - Simultaneous use with other products only after consultation with your doctor.

Care instructions:
Before cleaning the product, close the valve by turning it to OFF and remove the pump hose from the valve. Make sure that the valve does not come into contact with too much water, otherwise water can enter the air chambers. The textile compression bandage can be cleaned by hand with water and a mild detergent. The cooling pad and all plastic parts can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:
Polyamide (PA), Polyester (PES), PVC, Silicone
Gel: Propylene Glycol, Sodium Carboxymethylcellulose (CMC)

Storage instructions:
Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:
Disposal via household waste is possible.

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