DK

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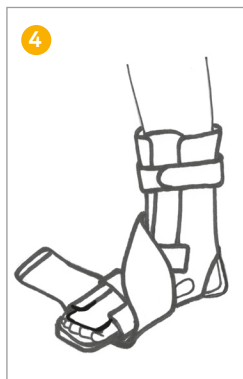
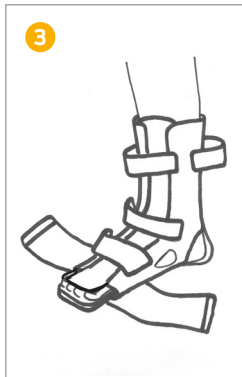
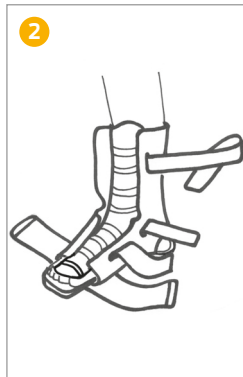
office@schaper-healthcare.com

www.schaper-healthcare.com

Size	XS	S	M	L	XL
Shoe size	32-35	35-37	37-40	40-43	43-46
Side	Universal				

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Intended use:

Foot brace to immobilise the ankle joint and fix the foot in dorsal flexion.

Indications:

Posttraumatic and postoperative immobilisation
Conditions in which it is necessary to keep the foot in dorsal flexion (e.g. for preventing horse foot)
Plantar fasciitis

Contraindications:

Currently unknown.

Application instruction:

Open all Velcro fasteners and straps and remove the middle section. Place the brace around the foot. Position the product so that the heel sits well in the heel area (1). Replace the middle section on the foot and lower leg and close the side sections of the brace using the Velcro fasteners on the middle section (2). First close the strap at the back of the foot and then the strap on the instep of the foot using the Velcro fastener. Then draw the topmost strap right around the brace and close it using the Velcro fastener (3). Pull the side straps tightly upwards, cross them over and fasten the straps above the ankle using the Velcro fasteners (4+5).

Application note / Caution:

- We recommend wearing a sock under the brace.
- The product is not suitable for walking.
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:

The product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product (if existent) should not be washed and must be removed if possible. These parts can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:

Polyamide (PA), PUR foam, Cotton (CO), Polyester (PES)

Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:

Disposal via household waste is possible.

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