DK

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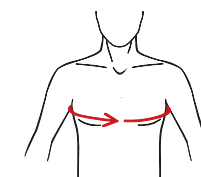
Size	XS	S	M	L	XL	XXL
Chest circumference (cm)	55-70	65-80	75-90	85-100	95-110	105-120

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Intended use:

Clavicle support to restrict movement and immobilise the shoulder girdle.

Indications:

Treatment of collarbone fractures (clavicle fractures).

Contraindications:

Currently unknown.

Application instruction:

Open the Velcro fasteners of the two shoulder straps (1). Place the back pad of the brace on the back, between the shoulder blades, and bring the straps forward over the shoulders. Draw the straps back under the arms (2). Thread the straps through the plastic loops on the back section. Pull the straps tight and close them using the Velcro fasteners (3+4). Adjust the height of the back pad by changing the length of the vertical strap using the Velcro fastener (5). Check the fit of the straps again and tighten them, if necessary, so that the orthosis sits correctly on the body and sufficient immobilisation is achieved. The straps should be positioned in such a way that they lie over the shoulders and under the arms without causing any constriction. Make sure the Velcro straps are not positioned in the armpits, otherwise the skin can be injured by the Velcro straps.

Application note / Caution:

- Users are advised not to wear the product directly on the skin, but for example over a shirt.
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:

The product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product (if existent) should not be washed and must be removed if possible. These parts can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:

Polyamide (PA), PUR foam, Cotton (CO), Polyester (PES), Polyethylene (PE)

Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:

Disposal via household waste is possible.

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