DK

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**Intended use:**

Elbow support for soft tissue compression.

**Indications:**

Soft tissue irritation in the area of the elbow

**Contraindications:**

Currently unknown.

**Application instruction:**

Fold the upper section of the brace outwards (1). Pull the brace over the forearm up to the elbow (2). Fold the upper part of the brace back again and pull the brace upwards if required (3). When putting on the brace, make sure it does not wrinkle in the area of the crook of the elbow.

**Application note / Caution:**

- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

**Care instructions:**

The product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product (if existent) should not be washed and must be removed if possible. These parts can be cleaned with a damp cloth. Close existing Velcro faste-ners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is com-pletely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

**Material composition:**

Polyamide (PA), Elastodiene (ED), Elastane (EL), Thermoplastic polyurethane (TPU). The product contains natural rubber latex and may cause allergic reactions

**Storage instructions:**

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

**Disposal note:**

Disposal via household waste is possible.

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