DK

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Polyester (PES), PUR-Schaumstoff, Polyamid (PA), Polyoxymethylen (POM)

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Intended use:

Shoulder joint orthosis for immobilization of the shoulder joint.

Indications:

Injuries to the rotator cuff, Shoulder luxation, Immobilization for humeral head fractures, Post-traumatic and postoperative immobilization

Contraindications:

Currently unknown.

Application instruction:

Place the wide chest strap below the chest and close the Velcro (1). Close the strap around the upper arm of the injured shoulder using the Velcro fastener (2). Move the shoulder strap forward behind the back over the healthy shoulder and place the injured shoulder's hand in the pocket (3). Make sure that the plastic buckle in the back is positioned in the upper part of the chest strap and that the Velcro fasteners of the shoulder strap are off the body (4). If necessary, thread the shoulder strap completely, place the strap correctly over the healthy shoulder and fix the strap back to the buckle using the Velcro fastener. Adjust the length of the shoulder strap with the help of the Velcro fastener and position the shoulder pad so that it rests well over the healthy shoulder (5). Adjust the shoulder strap so that the arm of the injured shoulder is positioned approximately at right angles. Fix the handbag with the help of the Velcro strap on the chest strap (6). If the problem arises that the chest strap in the back area is pulled too much upwards from the arm strap, it may be either that the chest strap is applied too loosely, or the length of the arm strap has been set too short. If this is the case with you, re-place the chest strap around the body, tighten it and close the Velcro fastener or adjust the strap to a longer length. However, when adjusting the shoulder strap, make sure that the arm is positioned approximately at a right angle.

Application note / Caution:

- We recommend wearing the product over a shirt.
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, Lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:

The product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product (if any) should not be washed and must be removed if possible. These parts can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:

Polyester (PES), PUR foam, Polyamide (PA), Polyoxymethylene (POM)

Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:

Disposal via household waste is possible.

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