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EN

Intended use:
Arm sling for light immobilization and relief of the shoulder joint.

Indications:
Immobilization and relief for post-traumatic pains
Postoperative support of the shoulder

Contraindications:
Currently unknown.

Application instruction:
Take the end of the arm loop to which the Velcro is firmly sewn and shape this end into a loop (1). The loop should be large enough for your forearm. Place the forearm of the injured shoulder in the loop and guide the other end of the loop behind the back over the healthy shoulder forward (2). Place the sling around the wrist and pin the end on the arm loop so that the wrist is in a loop (3). Make sure that the arm is positioned at a right angle if possible and that you feel comfortable. The length of the arm loop can be shortened if necessary. To do this, open the Y-Velcro fastener, shorten the strap, and then tighten the Y-Velcro fastener (4).

Application note / Caution:
• Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
• This product is intended for use on a single patient only.
• If necessary, the help of a second person can be beneficial for the correct application of the product.
• Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
• The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
• Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
• Do not put on the product too tightly, as this may affect the blood circulation.
• If symptoms persist or worsen, please consult your medical doctor.
• Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
• If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
• Do not make any changes to the product.
• Simultaneous use with other products only after consultation with your doctor.

Care instructions:

The textile parts of the product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:

Polyamide (PA), PUR foam

Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:

Disposal via household waste is possible.

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