DK

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office@schaper-healthcare.com

www.schaper-healthcare.com

Size	XS	S	M	L	XL
Shoe size	33-35	36-38	39-41	42-44	45-46
Side	Universal				

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Intended use:

The forefoot relief shoe is designed to provide relief in the area of the forefoot.

Indications:

Wounds and injuries in the forefoot area
Postoperative

Contraindications:

Currently unknown.

Application instruction:

Open the strap of the shoe and remove the front velcro. Fold the side parts outwards. Place your foot in the open shoe (1). Ensure that the foot is securely positioned in the shoe and that the desired pressure relief is achieved. Fold the side parts shut. First attach the wide part of the Velcro strip to the shoe and then close the two thinner parts of the Velcro strip with Velcro fasteners (2). Finally, close the belt with Velcro (3).

Application note / Caution:

- While wearing the shoe, you must not drive a vehicle.
- If the sole of the shoe is worn or damaged, the product should no longer be used.
- Do not wear the shoe directly on the skin.
- Walk slower than normal and be especially careful when walking on uneven paths and when climbing stairs.
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that no fabric parts are damaged by existing Velcro fasteners.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:

The product can be cleaned by hand with a mild detergent. Plastic and metal parts of the product (if existent) must not be washed and must be removed if possible. These parts can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to prevent damage to other laundry items. Leave to air dry. Avoid direct heat sources and do not machine dry or iron. Do not use chemical cleaning products, bleaches, solvents or fabric softeners. Fatty or acidic substances, ointments or lotions can damage the material.

Material composition:

TPE, Polyester, PU foam, Nylon, EVA

Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:

Disposal via household waste is possible.

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