Schaper Healthcare GmbH
Vogelweiderstraße 146 | 4600 Wels | Austria
office@schaper-healthcare.com
www.schaper-healthcare.com

Version V01/2021/08

DK

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Fingerskinne til immobilisering af fingerens mellemled (PIP-led) og fingerens endeled (DIP-led).

Indikationer:

Ledskader
Frakturer i fingerens endeled (distale phalanx) og/eller fingerens midterled (mediale phalanx)
Posttraumatisk immobilisering

Kontraindikationer:

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Intended use:
Finger splint for the immobilisation of the proximal interphalangeal joint (PIP joint) and the distal interphalangeal joint (DIP joint).

Indications:
Injuries to the joints
Fractures of the distal interphalangeal joint (distal phalanx) and / or of the proximal interphalangeal joint (medial phalanx)
Posttraumatic immobilisation

Contraindications:
Currently unknown.

Application instruction:
Slightly pull the finger splint apart (1). Place the finger into the splint and mould the aluminium part to the finger by bending it (2+3). When moulding the splint, ensure that sufficient fixation of the finger is achieved and that the metal and foam edges do not create any pressure points. The finger splint can be fixated with a Velcro® strap or an adhesive bandage for better support (4+5).

Application note / Caution:

- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:
The product can be cleaned with a clean, damp cloth. Avoid direct heat sources and do not machine dry or iron. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:
Aluminium, PE foam

Storage instructions:
Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:
Disposal via household waste is possible.

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