2

4

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Intended use:
Shoulder support for strain relief and stabilisation of the shoulder joint.

Indications:
Soft tissue irritation in the shoulder area

Contraindications:
Currently unknown.

Application instruction:
Pull the brace over the upper arm and the affected shoulder (1+2). Pull the support strap past the back of the head and pass the healthy arm through the support strap (3). Make sure the underarm pad rests in a comfortable, pressure-free position. Pass the chest strap around the body above the chest, thread the ends through the plastic loops on the underarm pad and close the strap using the Velcro fasteners (4). The position of the inner pad can be changed using the Velcro fasteners. Place the pad so that it encloses the humeral head well.

- Application note / Caution:**
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
 - This product is intended for use on a single patient only.
 - If necessary, the help of a second person can be beneficial for the correct application of the product.
 - Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
 - The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
 - Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
 - Do not put on the product too tightly, as this may affect the blood circulation.
 - If symptoms persist or worsen, please consult your medical doctor.
 - Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
 - If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
 - Do not make any changes to the product.
 - Simultaneous use with other products only after consultation with your doctor.

Care instructions:
The product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product (if existent) should not be washed and must be removed if possible. These parts can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:
Polyamide (PA), Elastodiene (ED), Elastane (EL), Silicone, Thermoplastic polyurethane (TPU), Polyester (PES)
The product contains natural rubber latex and may cause allergic reactions.

Storage instructions:
Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:
Disposal via household waste is possible

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