DK

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Intended use: Ankle orthosis for stabilization of the ankle joint.

Indications: Instability of the ankle, Capsule and ligament injuries, such as distorsion (sprain), contusion (bruise), etc., Pronation and supination trauma, Postoperative for stabilization after ligament suture

Contraindications: Currently unknown.

Application instruction: Open the circular belts of the splint completely. Adjust the heel strap to the foot so that the side plastic shells are well attached to the foot. To do this, open the Velcro fasteners on the inner side of the shells and adjust the appropriate length (1). Place your foot in the splint and attach the plastic shells (2). Make sure that the shells are located on the side of the lower leg and adapt well to the ankle. Now first guide the lower belt and then the upper belt around the splint and close the straps using the hook-loop fasteners (3+4). If the belts are too long, the protruding part can be shortened.

Application note / Caution:

- We recommended wearing a sock or stocking under the splint.
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor

Care instructions: The textile parts of the product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition: Polyester (PES), PUR foam, Polypropylene (PP), Polyamide (PA)

Storage instructions: Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note: Disposal via household waste is possible.

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