

**Schaper**  
HEALTHCARE

DE EN FR NL DK SE

## Schapfix TLSO

Model: SCSFT

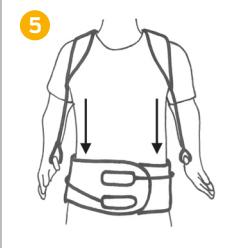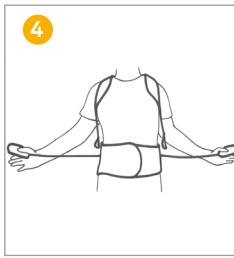**DK**

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Polyester (PES), PUR-Schaumstoff, Polyethylen (PE), Aluminium, Polyoxymethylene (POM), Polyamid (PA), PVC

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**Intended use:**

Orthosis for straightening and relieving strain on the lumbar spine and thoracic spine.

**Indications:**

Dorsalgia  
Round back with shoulder malposition  
Kyphosis  
Scheuermann's disease  
Osteoporosis  
Stable, osteoporotic vertebral body fractures

**Contraindications:**

Currently unknown.

**Application instruction:**

To ensure that the orthosis fits correctly, you can individually shape the aluminium support in the back section and adjust the height of the back section. To do this, position the back section at the desired height and fix it using the Velcro fastener (1). Open the chest strap and the front fastener of the orthosis and release the pull cords. Put the orthosis on and close the bottom two fastener sides using the Velcro fastener (2+3). Place the fastener sides evenly over one another. Use the hand loops for easier donning. Position the orthosis so that the back section is centred in the lower back area. Now take the two lateral pull cords and pull them forward with an even traction (4). Once the desired stabilisation has been achieved, fasten the pull cords using the Velcro fasteners. Pull the shoulder straps tight (5). Close the quick-release fastener on the chest and pull it tight (6+7). To take the product off, first release the lateral pull cords and then open the front fastening. If necessary, the quick-release buckles on the shoulder straps can be used to put on and take off the orthosis. To do this, open the quick-release buckles by pressing the button and releasing the buckle. Make sure that the button fully engages again when fastening.

**Application note / Caution:**

- Users are advised not to wear the product directly on the skin, but for example over a shirt.
- Only change the settings of the product after consulting your attending physician.
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

**Care instructions:**

The textile parts of the product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

**Material composition:**

Polyester (PES), PUR foam, Polyethylene (PE), Aluminium, Polyoxymethylene (POM), Polyamide (PA), PVC

**Storage instructions:**

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

**Disposal note:**

Disposal via household waste is possible.

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