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EN

Intended use:
Lower leg foot orthosis for immobilization of the lower leg foot area.

Indications:
Post-traumatic / Preoperative / Postoperative immobilization of the lower leg foot area
Severe ankle distortions
Stable foot and ankle fractures
When using Achilles tendon wedges / heel wedges: Achilles tendon rupture

Contraindications:
Currently unknown.

Application instruction:

Do not wear the orthosis directly on the skin, but wear a sock or stocking over your foot. Open all Velcro fasteners and straps and remove the front cover. Do not remove the padding from the shell. Open the padding and fold the side panels and the sewn-on crack protection outwards. Place the foot and lower leg in the splint (1). Position the foot so that the heel sits at the back of the shell. Make sure you have a good fit. Place the sewn-on rist pad around the back of the foot. Close the padding using the Velcro fasteners and ensure a wrinkle-free fit (2). For a better fit and for further pressure relief, place additional cushions between the shell and the shell padding or the shell padding and the foot if necessary. If necessary, attach the toe protection to the padding below and above the toes using the Velcro fastener (3). Position the toe guard so that you do not feel any pressure points in the area of the toes. Use the Velcro fasteners to reattach the front cover to the center of the padding (4). When attaching the front cover, make sure that the valve is not bent and that the valve and pump are properly seated in the openings and are not clamped. Close the straps of the splint using the hook-loop fasteners. Start with the strap on the ankle. Then close the strap over the back of the foot, then the lower leg strap and finally the upper lower leg belt (5).To fill the air chambers, first turn the valve clockwise to „Inflate“. Then use the black pump to pump as much air as you can until the desired stabilization is achieved. To drain air, turn the valve counterclockwise to „Deflate“ (6).

Application note / Caution:

- If you feel pressure through the hose running in the padding, place a soft padding between the pressure point and the hose.
- While wearing the orthosis, you are not allowed to operate a vehicle.
- In case of wear or damage to the sole of the shoe, the product should no longer be used.
- Walk slower than normal and be especially careful when walking on uneven paths when climbing stairs.
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that no fabric parts are damaged by existing Velcro fasteners.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:
Before cleaning the product, close the valve by turning it to "Inflate". Make sure that the pump and the valve do not come into contact with water, otherwise the water can enter the air chambers. The textile upholstery can be cleaned by hand with water and a mild detergent. The plastic shell and all plastic and metal parts of the product can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items.

Material composition:
EVA foam, Polyamide (PA), PUR foam, Polypropylene (PP)

Storage instructions:
Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:
Disposal via household waste is possible.

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