DK

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Intended use:

Orthosis for stabilization and immobilization of the cervical spine.

Indications:

Postoperative immobilization, Post-traumatic conditions of the cervical spine (sling-shot trauma, distorsion, etc.), Diseases that require a rough immobilization of the cervical spine, Pain conditions in the area of the cervical spine

Contraindications:

Currently unknown.

Application instruction:

The product consists of a front part (sticker with „FRONT“) and a neck part (sticker with „BACK“). Place the neck part around the neck and the front part around the neck (1+2). Position the two parts so that the front part is sideways above the neck part. Make sure that the chin rests well on the chin rest. Close the Velcro fasteners evenly on both sides to ensure a stable fit (3+4).

Application note / Caution:

- The cervical support must not be removed without consultation with your attending physician.
- Change the settings of the product only after consultation with your attending physician.
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:

The textile parts of the product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:

EVA, polypropylene (PP), Polyamide (PA), Cotton (CO)

Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:

Disposal via household waste is possible.

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