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Intended use:

Knee joint orthosis for stabilising and guiding the knee joint with the option of restricting the range of movement (extension and flexion) of the joint.

Indications:

Moderate instability of the knee joint
For treatment of the knee after ligament injuries
Meniscus injuries
Postoperative treatment

Contraindications:

Currently unknown.

Setting the extension/flexion limit:

You can adjust the desired range of motion on the joints of the rail. To do this, open the joint lid (1). Remove the boundary pins and insert them into the appropriate degree setting (2). Make sure that both boundary pins are placed in the same degree setting to adjust the extension/flexion. Close the joint lid completely again.

Flexion limit: 0°, 15°, 30°, 60°, 90°
Extension limit: 0°, 15°, 30°, 60°, 90°

Application instruction:

If you can, sit on a chair and bend the leg slightly to make it easier to put the brace on. Open all Velcro fasteners and straps. Place the brace around the leg (3). Position the joints of the brace on the side of the knee so that the centre of the joint is at the level of the upper edge of the kneecap. Close the lower leg padding and then the thigh padding using the Velcro fasteners (4+5). When closing the padding, make sure the kneecap is in the recess of the padding. Close the straps of the brace using the Velcro fasteners. Start with the top strap on the lower leg (6). Then close the bottom strap on the lower leg, then the bottom thigh strap and, finally, the top thigh strap (7+8).

Application note / Caution:

- When using the knee brace, the limit pins must always be placed in the joint and must never be completely removed.
- To prevent any overstretching of the knee joint, a limit pin must always be placed at the 0° extension setting, unless your doctor specifies a different setting.
- It is generally recommended not to wear the knee orthosis directly on the skin.
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that no fabric parts are damaged by existing Velcro fasteners.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Care instructions:

The textile parts of the product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:

Airprene neoprene (CR), Polyamide (PA), Iron, Aluminium, Polypropylene (PP)

Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:

Disposal via household waste is possible.

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